

## Mirage Micro™ for Kids

NASAL MASK

### Fitting Guide



- Turn the MicroFit dial to position 24, where the forehead support is fully extended.



- Disconnect one headgear clip, place the mask on the face and pull the headgear over the head.



- Bring the lower straps below the ears.
- Connect the lower headgear clip ensuring it clicks into place.



- Unfasten the upper Velcro™ tabs, pull the headgear straps evenly until comfortable, then reattach the Velcro.
- Do not overtighten the straps.



- Repeat Step 4 with the lower headgear straps.
- Connect one end of the air tubing to the swivel and the other end to the therapy device.
- Turn on the therapy device.



- While lying down, turn the MicroFit dial until the mask is evenly and comfortably positioned on the face.
- If any air leaks are felt, readjust the dial and/or headgear.

 For details, refer to the *Mirage Micro for Kids Nasal Mask User Guide*.