

# Mask/Device Compatibility List

This document specifies the masks compatible with ResMed's positive airway pressure devices and the mask setting, if one is required. For full details regarding the correct use of these devices, please refer to the specific User Guides.

*Note: Not all products are available in all regions.*

## Full Face Masks

Device	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her	Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Ultra Mirage	Hospital Full Face
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 Escape™ / S9 Escape™ Auto	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
C-Series Tango™	No setting required. Consult your clinician before changing to a different mask type.					
S7 Elite™	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Vantage™	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Compact™	No setting required. Consult your clinician before changing to a different mask type.					
S8 Elite™ / S8 Elite™ II	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Escape™ / S8 Escape™ II	No setting required. Consult your clinician before changing to a different mask type.					
S8 AutoSet™ II	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III ST-A	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP Adapt SV™	FULL FACE	FULL FACE	FULL FACE	FULL FACE <sup>1</sup>	FULL FACE	FULL FACE
VPAP™ Adapt (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Adapt (S9 ASV)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Auto 25	MIR FULL	MIR FULL	MIR FUL	MIR FULL	MIR FULL	MIR FULL
VPAP™ Auto (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP Malibu™	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VPAP™ S / VPAP™ ST	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ S (S9) / VPAP™ ST (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ ST-A (S9 iVAPS)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ COPD (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Tx	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
Stellar™	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VS Ultra™ / VS Integra™	✓	N/V	✓	✓	✓	✓
VS III™	✓	N/V	✓	✓	✓	✓

<sup>1</sup>SmartStop may not operate as described when using the mask with ResMed VPAP Adapt SV.

N/V = The mask has not been validated for this device.



Device	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage FX™ / Mirage FX for Her	Mirage™ SoftGel	Mirage Activa™ LT	Mirage Micro™	Ultra Mirage II	Mirage Activa™	Mirage Vista™	Hospital Nasal	Meridian™	Mirage Kidsta™	Mirage Micro™ for Kids	Pixi™
VS III™	N/V	N/V	N/V	✓	✓	✓	✓	✓	✓	N/V	✓	N/V	N/V

<sup>1</sup> SmartStop may not operate as described when using the mask with ResMed VPAP Adapt SV / Stellar.

N/V = The mask has not been validated for this device.

## Nasal Pillows

Device	Swift™ FX / Swift™ FX for Her	Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT / Swift™ LT for Her	Mirage Swift™ II
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Pillows	Pillows	Pillows	Pillows
S9 Escape™ / S9 Escape Auto™	Pillows	Pillows	Pillows	Pillows
C-Series Tango™	No setting required. Consult your clinician before changing to a different mask type.			
S7 Elite	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S8 AutoSet Vantage™	SWIFT	SWIFT	SWIFT	SWIFT
S8 Compact™	No setting required. Consult your clinician before changing to a different mask type.			
S8 Elite™ / S8 Elite™ II	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™ / S8 Escape™ II	No setting required. Consult your clinician before changing to a different mask type.			
S8 AutoSet™ II	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ III	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ III ST-A	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP Adapt SV™	N/V	N/V	N/V	N/V
VPAP Adapt (S9)	Pillows	Pillows	Pillows	Pillows
VPAP™ Adapt (S9 ASV)	Pillows	Pillows	Pillows	Pillows
VPAP™ Auto 25	SWIFT	SWIFT	SWIFT	SWIFT
VPAP Auto (S9)	Pillows	Pillows	Pillows	Pillows
VPAP Malibu™	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S / VPAP™ ST	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S (S9) / VPAP™ ST (S9)	Pillows	Pillows	Pillows	Pillows
VPAP™ ST-A (S9 iVAPS)	Pillows	Pillows	Pillows	Pillows
VPAP™ COPD (S9)	Pillows	Pillows	Pillows	Pillows
VPAP™ Tx	PILLOWS	PILLOWS	PILLOWS	PILLOWS
Stellar™	PILLOWS <sup>1</sup>	PILLOWS <sup>1</sup>	PILLOWS <sup>1</sup>	PILLOWS <sup>1</sup>
VS Ultra™ / VS Integra™	N/V	N/V	N/V	N/V

Device	Swift™ FX / Swift™ FX for Her	Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT / Swift™ LT for Her	Mirage Swift™ II
VS III™	N/V	N/V	N/V	N/V

<sup>1</sup>SmartStop may not operate as described when using the mask with Stellar.

N/V = The mask has not been validated for this device.

**Product**

Stellar  
VS III / VS Ultra / VS Integra  
All others



**Manufacturer**

**ResMed Germany Inc.** Fraunhoferstr. 16 82152 Martinsried Germany  
**ResMed Paris** 240 rue de la Motte 77550 Moissy-Cramayel, France  
**ResMed Ltd** 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia

**Distributed by**

**ResMed Corp** 9001 Spectrum Center Boulevard San Diego CA 92123 USA, **ResMed Ltd** 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia  
**ResMed Ltd** 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia  
**ResMed Corp** 9001 Spectrum Center Boulevard San Diego CA 92123 USA, **ResMed (UK) (for Ltd)** 96 Jubilee Ave Milton Park Abingdon Oxfordshire OX14 4RW UK

See [www.resmed.com](http://www.resmed.com) for other ResMed locations worldwide. Tango, S7, AutoSet, S8, S8 AutoSet Vantage, S8 Compact, S8 Elite, S8 Escape, S9, S9 AutoSet, S9 Elite, S9 Escape, VPAP, VPAP Adapt SV, VPAP, VPAP Tx, Malibu, Stellar, Mirage, Ultra Mirage, Mirage Quattro, Mirage Liberty, Mirage Micro, Mirage Vista, Meridian, Activa, Kidsta, Pixi, Swift and Mirage Swift are trademarks of ResMed Ltd. VS Ultra, VS Integra and VS III are trademarks of ResMed Paris SAS.

© 2013 ResMed Ltd. 1013348-Eng/15 2013-05